



# Auditory Outreach... A Provincial Resource Program

## Bone Anchored Hearing Systems (BAHS)

Listening checks and FM coupling are similar across the make and models of BAHS.

Oticon  
Ponto



Cochlear  
Intenso



Cochlear Divino  
coupled Phonak  
BAHA MLxi



Phonak  
BAHA MLxi



### Proper positioning of the sound processor


- Ensure that the processor is vertical, with the microphone inlet is on the bottom.

### How to couple the receiver and BAHS

- Line up the receiver marked with red or silver, with the input on the BAHS marked with red or silver. Use some force and attach.

### Listening Check

- Soft-band BAHS. With the BAHS on, gently place the soft band on your head with the BAHS situated on your forehead. Then cover your ears and listen to your voice and another person's speech.
- Implanted BAHS. With the BAHS on, couple it gently to one end of the test rod. Place the other end on your mastoid (the bone behind your ear), and cover your ears. Listen to your voice and another person's speech.

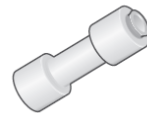

Test Rod

### Trouble-Shooting Tips

Symptom	Possible Issues and Things to Try
No Sound	Worn out battery? Change the battery.
Intermittent or Reduced Signal	Dirt in the coupling? Clean the processor with a dry cloth or baby wipe. Moisture? Open the battery door, and let the sound processor dry out. Worn out battery? Change battery.
Whistling	Sound processor is touching something? Check to make sure the sound processor does not touch anything, including the outer ear