



Auditory Outreach... A Provincial Resource Program

4351 Ontario Avenue, Powell River, BC V8A 1V3

Toll Free: 1-866-430-4327

Fax: 604.485.2886

www.auditoryoutreach.ca

Student: _____ Date of Birth: _____

School: _____ Teacher: _____

EasyLink+ Transmitter

Battery: Rechargeable (ensure system is turned off to charge)



To Wear: Loop the antennae over the neck
Do not loop or knot the lavalier antennae for maximum operating range

On/Off: Press and hold the power button until the channel appears on the LCD screen

To Change Channel: Use a pen tip, press and hold the channel changer button
The channel on the screen will blink and scroll through a channel list
Release button at desired channel, wait until channel stops blinking
Turn Easylink+ off, then on, within 8 inches of the receiver(s)

Did the student hear beeps?
Is the signal clear? If so, **contact an Audiologist to program this channel as default for the receiver.**

Trouble Shooting

Issue	Try
No Sound	<ol style="list-style-type: none"> 1. Is the EasyLink+ charged? 2. Ensure the EasyLink+ and receiver(s) are on the same channel. Sync the Easylink+ and receiver by turning the Easylink+ off, then on, within 8 inches of the receiver. The receiver must be on.
Interference/Static	<ol style="list-style-type: none"> 1. Change the channel; contact Audiologist to co-ordinate the receiver's default channel.
Insufficient Operating Range	<ol style="list-style-type: none"> 1. Un-knot the antennae.

...Please contact Auditory Outreach if the above steps do not resolve the issue.