



Auditory Outreach... A Provincial Resource Program

4351 Ontario Avenue, Powell River, BC V8A 1V3

Toll Free: 1-866-430-4327

Fax: 604.485.2886

www.auditoryoutreach.ca

Student: _____

Date of Birth: _____

School: _____

Teacher: _____

Phonak Inspiro Transmitter

Battery: Rechargeable



To Wear: Clip the Inspiro to your belt or shirt pocket.
 Attach the EasyBoom microphone to the EasyBoom adapter cable.
 Pull the microphone over your head, with the flex wire resting on your ears and the black nubs pressing gently on your temples.
 Switch the Inspiro on.

Trouble Shooting

Issue	Things to Try
No Sound	1. Restart the Inspiro. 2. Check the battery status. 3. Check the EasyBoom microphone connection to the EasyBoom adapter cable; has it been plugged in fully?
Intermittent Signal	1. Check the battery status. 2. Check the EasyBoom microphone connection to the EasyBoom adapter cable.
Static	1. Check EasyBoom microphone placement; is it rubbing on anything?

...Please contact an Auditory Outreach if the above steps do not resolve the issue.