


Digital Remote Microphone Set-Up Tips


Hearing Device: Nucleus 6 CI Processor

Equipment: Roger Inspiro Transmitter with iLapel Microphone, Roger X Receiver & Euro-Adaptor

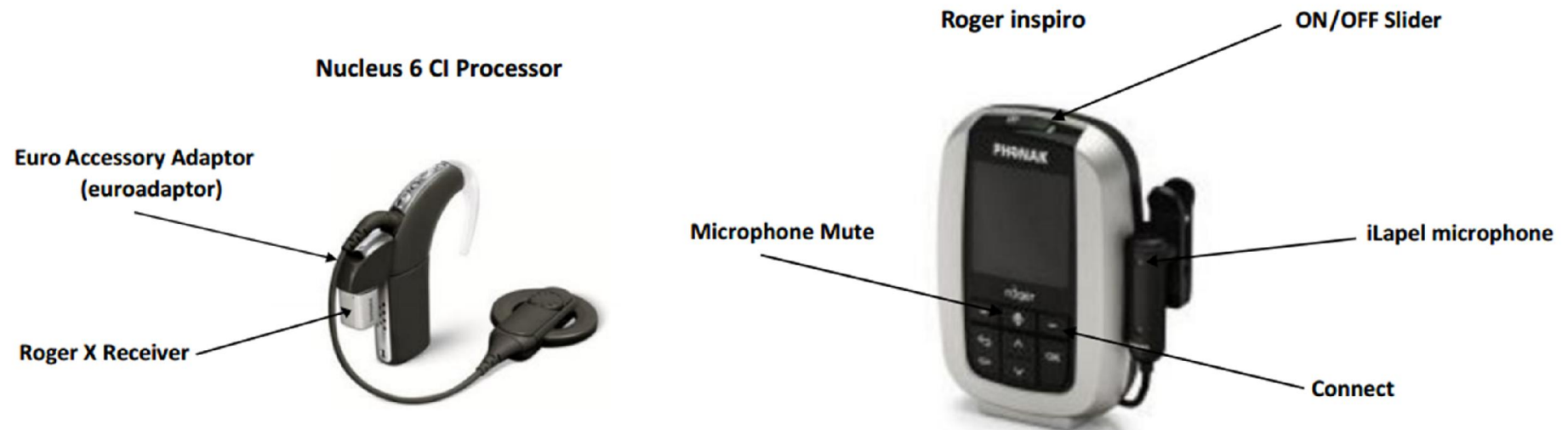

Attaching the Roger X receiver on the CI:

1. Connect the **Roger X receiver** to the **euroadaptor** by aligning the biggest pin on the receiver with the biggest hole in the adaptor. Be sure the connection is tight by pushing them together until it **clicks**.
2. The CI processor should be turned **ON** first before you plug in the receiver.
3. Lift up the accessory port cover on the CI processor with a small screwdriver (or by twisting the battery module).
4. Insert the **receiver+euroadaptor unit** into the accessory port.
5. Check that there is a blue (N6) flickering light on the earhook of the processor, which indicates the Roger X receiver has been detected.

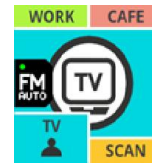

6. FM Auto or FM Standby will also show on the home screen of the remote assistant.

***Connecting to the Roger Inspiro Transmitter:**

- 7. Turn ON the Roger Inspiro by sliding the top slider until green appears.
- 8. Holding the Roger Inspiro within **3 inches** of EACH powered-ON Roger X receiver, press Connect (right soft key with dash).
- 9. Test the connection by asking if the student hears a tone.

** A new Roger receiver needs to be connected to its transmitter only once. The connection holds until the receiver is connected to another Roger transmitter.*

Toggling IN and OUT of FM mode:


This is recommended for more mature and experienced users only. To toggle OUT of FM when the receiver is connected, either 1) press the top button on the N5/N6 processor, or 2) press the left button on the side of the remote assistant.



IMPORTANT: *Be sure to toggle back into FM mode when instruction resumes!*

Using the Roger Inspiro:

1. The lapel microphone should be clipped at 20 cm directly below the teacher's lips. The directional microphones should stay upright as much as possible.
2. The microphone cable acts as the antenna, and should be unwound all the way during use.

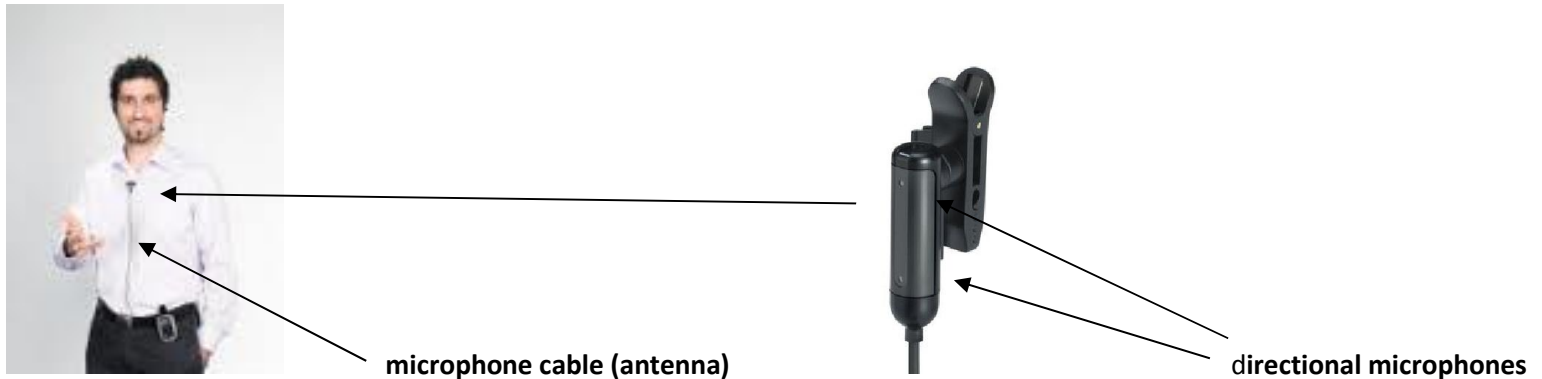

Monitoring by Behavioural Check:

1. From a distance of at least 3 metres away, speak into the transmitter microphone while covering your lower face. Ask the student to do things like “point to the window”, “put up your hand” etc...
2. If the system works, the student should respond immediately (even if it's just looking up from what s/he was doing and searching for you).

During brief teaching breaks (e.g. 15-min quiet study or speaking to individual students):

1. MUTE the microphone by pressing the microphone mute button once.
2. IMPORTANT: Remember to UNMUTE the microphone when instruction resumes! Do this by pressing the same button once.

During recess, lunch, or between classes:

Turn OFF the Roger Inspiro Transmitter by moving the top slider until the green part disappears.

At the end of the day:

Charge the transmitter at a safe, dry place. It takes 2 hours to fully charge it. A fully charged Roger Inspiro operates for about 8 hours.