



Auditory Outreach... A Provincial Resource Program

4351 Ontario Avenue, Powell River, BC V8A 1V3

Toll Free: 1-866-430-4327

Fax: 604.485.2886

www.auditoryoutreach.ca

Student: _____ Date of Birth: _____

School: _____ Teacher: _____

iSense Micro Receiver


Battery: size 312 (Battery should be changed every week)



Fit: The black and silver battery compartment sits behind the ear.
 The flex wire bends over top of the ear.
 The speaker rests in the ear canal.

On/Off: Open the battery door to turn the iSense off.
 Close the battery door to turn the iSense on.

Trouble Shooting

Issue	Try
No Sound	1. Re-start the iSense (turn the iSense off then on) 2. Replace the battery. Ensure that the battery are inserted properly. 3. Replace the wax filter. <div style="display: flex; align-items: center; justify-content: center;"> <div style="border: 1px solid black; padding: 5px; margin-right: 10px;">To insert new wax filter</div>  <div style="border: 1px solid black; padding: 5px; margin-left: 10px;">To remove old wax filter</div> </div> 4. Ensure the iSense and transmitter are on the same channel (use transmitter to do this - see transmitter cheat sheet)
Intermittent Signal	1. Replace the battery. Ensure that the batteries are inserted properly.
Interference/Static	1. Replace the battery. Ensure that the batteries are inserted properly. 2. Change the channel (use transmitter to do this - see transmitter cheat sheet)

...Please contact Auditory Outreach if the above steps do not resolve the issue.